



DENTIRO® Foam

Gentle care for sensitive materials



- ▶ **Disinfecting care product**
- ▶ **Gentle formulation**
- ▶ **For sensitive materials**

DENTIRO® Foam is a ready-to-use foam for the disinfection, cleaning, and care of medical devices with sensitive surfaces (e.g. plastic components), including treatment chairs, examination tables and similar equipment. The gentle formulation protects sensitive materials such as synthetic leather. DENTIRO® Foam is easily biodegradable.

Use



Fill DENTIRO® Foam into foamer bottle. Disperse foam onto the areas to be disinfected. Use a dry single-use cloth to spread foam evenly and leave to act. Adhere to exposure times. Dry surface with a clean single-use cloth. Use once or twice daily.

Scope of Application



- ▶ For the disinfection, cleaning, and care of medical equipment with sensitive surfaces
- ▶ Particularly suitable for alcohol-sensitive items made of plastic, treatment chairs, and examination tables, as well as medical devices made of sensitive materials such as PVC

Precautions



R10 Flammable. S2 Keep out of the reach of children. S24 / 25 Avoid contact with skin and eyes. S26 In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Use disinfectants safely. Always read label and product information before use.

Delivery Forms



200 ml foamer flask, 2 liter

Effective Spectrum / Exposure Times

Bactericide	10 minutes
Fungicide (<i>C. albicans</i>)	10 minutes
Hepatitis B / HIV (1)	5 minutes

1) HBsAg antigen test

Complies with DGHM Standard Methods



Composition in 100 g

10 g 2-Propanol, 1.2 g Phenoxyethanol, 0.2 g Quaternary ammonium compounds.

