



DENTIRO® Wipes

The 'Classic' in 2 sizes and 3 fragrances



- ▶ **Excellent material compatibility**
- ▶ **Residue-free drying**
- ▶ **2 sizes / 3 fragrances**

DENTIRO® Wipes are ready-to-use and pre-moistened for the residue-free quick disinfection and cleaning of small surfaces of non-invasive medical devices (e.g. operating tables, suction hoses, hand-pieces, contra-angles, operating light handles, and medical equipment made of aluminum, copper, silicon, rubber, and plastics such as PVC and acrylic glass). The high level of saturation allows for a simple and efficient use of each single DENTIRO® Wipe. A low alcohol content of less than 50% achieves very good material- and skin compatibility and reduces the potential of allergy. DENTIRO® Wipes are aldehyde-free and easily biodegradable. To increase saturation, all DENTIRO® products, ISORAPID® Liquid, and ISORAPID® Spray can be used.

Use



Before use, cut open the refill bag. Pull the first wipe through the star-shaped opening in the lid and place the lid on wipes box. Pull the wipes sideways from the wipes box. Completely moisten the surface or object to be disinfected and leave to act. Adhere to exposure times. If necessary, dry the surface or object with a clean single-use cloth. Use on alcohol compatible materials only. Close wipes box firmly after each use. Test before use on sensitive materials by applying to a small inconspicuous area.

Scope of Application



- ▶ For the fast-acting wipe disinfection and cleaning of small surfaces of medical devices, e.g. operating tables, suction hoses, hand-pieces, contra-angles, operating light handles, and medical equipment made of aluminum, copper, silicon, rubber, and plastics such as PVC and acrylic glass

Precautions



R10 Flammable. S2 Keep out of the reach of children. S16 Keep away from sources of ignition – No smoking. S24 / 25 Avoid contact with skin and eyes. Keep in a cool and dry place. Use disinfectants safely. Always read label and product information before use.

Delivery Forms



Box of 120 wipes (14.5 x 20 cm).
Fragrances: Classic, floral, lemon
12 x refill bags of 120 wipes.
Fragrances: Classic, floral, lemon

Jumbo box of 100 wipes (14.5 x 40 cm).
Fragrances: Classic, lemon
12 x Jumbo refill bags of 100 wipes.
Fragrances: Classic, lemon



Effective Spectrum / Exposure Times

| | |
|---|------------|
| Bactericide / MRSA (1) | 60 seconds |
| Fungicide (<i>C. albicans</i>) | 60 seconds |
| Tuberculocide (<i>M. terrae</i>) | 60 seconds |
| Enveloped viruses (incl. HBV / HIV) (2) | 30 seconds |
| BVDV / Hepatitis C (3) | 15 seconds |
| Coronavirus | 30 seconds |
| Vaccinia virus | 30 seconds |
| Influenza A virus | 15 seconds |
| Herpes simplex virus | 15 seconds |

1) Methicillin-resistant *Staphylococcus aureus*; 2) Combined vaccinia virus / BVDV surrogate virus tests according to RKI / DVV guidelines; 3) BVDV surrogate virus test

Complies with European Standards and EN 1040, EN 1276, EN 1275, EN 1650, EN 13697, EN 14348 and RKI / DVV guidelines



Composition in 100 g

31.3 g Ethanol, 16.7 g 2-Propanol.